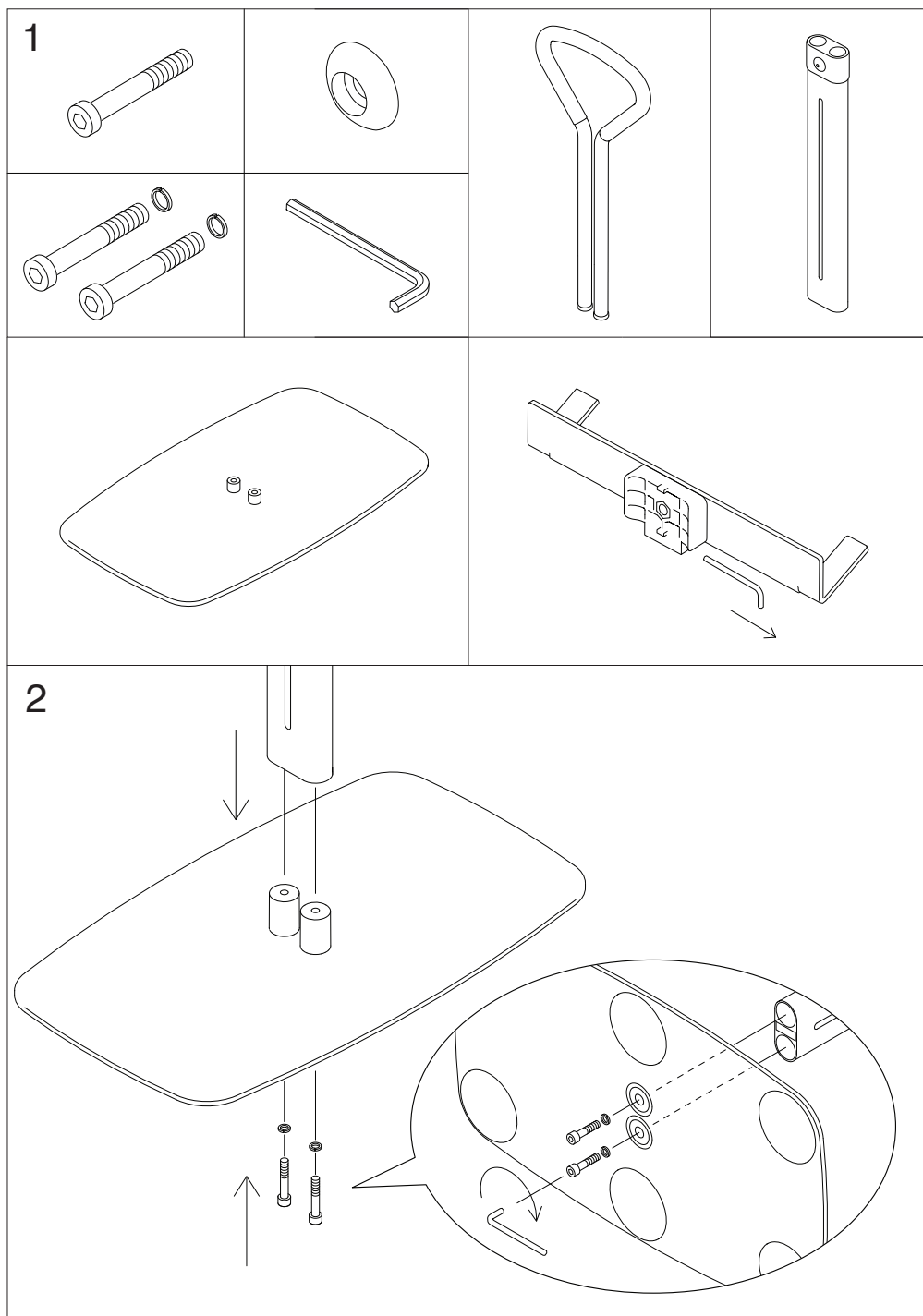
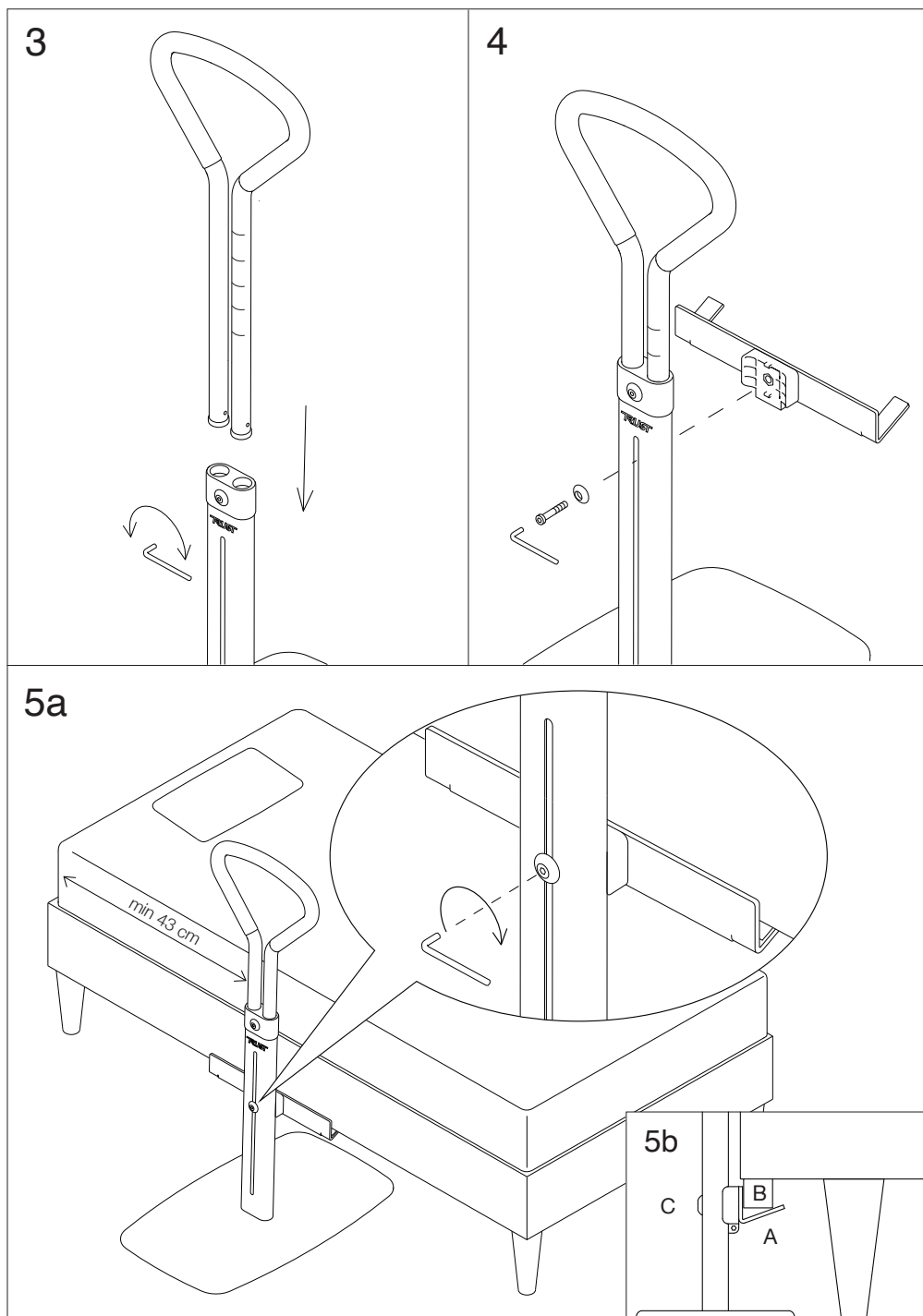
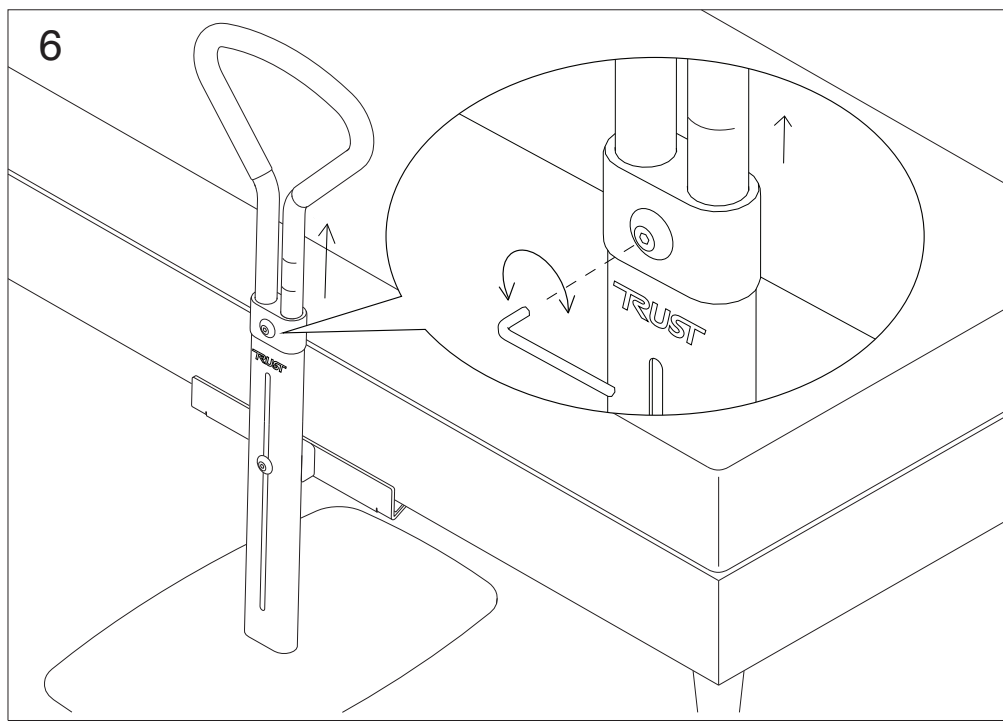
Let's Get Up

Uppresningstöd Let's Get Up

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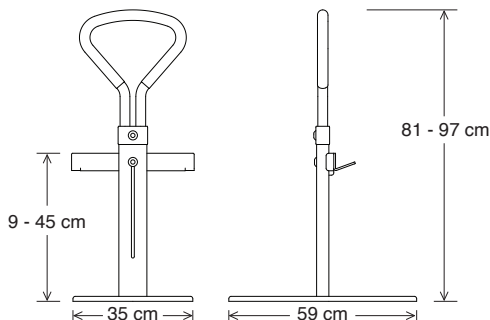
Let's Get Up – Stand Support

This manual contains important information about the Let's Get Up stand support. Please read the instructions carefully before use and keep them for future reference.

Intended use: Let's Get Up is intended to help users get in and out of bed and can be used in both home and care environments. The product may only be used on beds with a stable frame, where the mattress does not collapse at the edge under load.

The product must not be used in any way other than intended. Trust Care accepts no responsibility if the product is used, modified, or assembled in any way other than as stated in this user manual. Regularly check the stability and strength of the product and that the screws are tightened!

Target group: The product is intended for adult users who have sufficient physical strength, balance, and cognitive ability to understand safe use.

Contraindication: The product must not be used on beds with a soft or unstable mattress edge where the mattress collapses significantly under load, or on beds without a fixed frame.

This product is not suitable for persons who:

- lack sufficient muscle strength in the arms or who have significant balance difficulties
- have cognitive impairment that affects safe use
- are unable to follow instructions

Incident reporting: Incidents or suspected malfunctions must be reported immediately to the manufacturer Trust Care AB, and serious incidents also to the competent authority in the member state where the user is located.

Assembly: The stand-up support is equipped with an adjustable frame bracket and must only be installed on compatible beds with a stable frame.

1. Unpack all parts. The hex key for assembly and adjustment is stored in the frame bracket for easy access.

2. Attach the pole to the floor plate using the two screws from underneath, tighten securely with the hex key.

3. Loosen the screw at the top of the pole slightly by turning it to the left. Insert the handle. Then secure the handle by turning the screw to the right.

4. Attach the frame bracket to the pole.

5a. Place the support flush against the bed — the product must be mounted centrally along the long side of the bed. The distance between the head of the bed and the Let's Get Up must be at least 43 cm to minimise the risk of entrapment.

5b. Attach the frame bracket under the bed. Ensure that the brackets (A) are flush against the bed frame (B). Tighten the screw (C) and make sure it is securely fastened.

6. Adjust the height of the handle by loosening the screw slightly. Move the handle to the desired height using the markings below the numbers on the handle tube. Tighten the screw and make sure the metal plate on the back clicks back into place.

Ensure that the frame bracket is correctly fitted before use and check that the locking screws for both the frame bracket and handle are properly tightened.

Check that the stand-up support is positioned flush against the bed and mattress — if a gap occurs, there is a risk of entrapment injuries.

Maintenance: Regular safety checks ensure safe function and trouble-free use. Check that no parts are damaged. Repairs may only be carried out by qualified personnel. Only original accessories and spare parts from Trust Care may be used. Do not remove any components from the bed raiser and do not use any accessories that have not been approved by the manufacturer, as otherwise the safe function of the product can no longer be guaranteed.

Do not use the bed stand-up support in a particularly hot or humid environment. The product must be stored in a dry environment at room temperature. For hygiene reasons, regular cleaning of the product is recommended. Cleaning can be done with warm water and if necessary with soapy water, pH value 5–9, or commercially available cleaning agents. If needed, a surface disinfectant, e.g. isopropanol 70%, may be used.

Reconditioning instructions: When reconditioning, the product must be inspected and worn or damaged parts replaced. Older screws with protruding screw heads must be replaced with new ones. The product can be disinfected in appropriate equipment at a maximum temperature of 85°C for no more than 3 minutes.

Recycling: The product must be disassembled and sorted by material. Aluminium parts and steel parts are to be disposed of as metal scrap, plastic parts sorted as plastic or combustible waste in accordance with local regulations. Take to a recycling centre.

Warranty: We offer a two-year warranty. The warranty does not cover wear parts such as the anti-slip rubber on the frame brackets and under the floor plate, as well as the handle rubber.

ENGLISH

Safety instructions

Correct assembly

After installation, check that the support stand is positioned tight against the bed and mattress without any gaps, and that the frame brackets are flush against the bed frame and the screws securely tightened. See figures 5a, 5b in the manual.

Entrapment risk

If the product is not assembled in accordance with this manual, an entrapment risk may arise if a gap is created between the bed and the stand-up support. Hands and fingers must only be placed on the foam-padded handle. The product contains openings where fingers may become trapped if placed incorrectly.

Check for any gap

If a gap occurs between the bed and the stand-up support, the risk of entrapment injuries increases with the size of the gap. The distance between the mattress and the stand-up support must be less than 4 cm under load, when the user is sitting on the edge of the bed. A check must be carried out at installation and regularly thereafter.

Correct use

The product is not intended to bear the user's full body weight but must only be used as support when standing up. The product has been tested for a maximum user weight of 150 kg.

User risk assessment

Before the stand-up support is installed, an assessment must be made of the user's physical and cognitive capabilities. A risk assessment must be carried out by healthcare personnel or another competent person prior to installation. Limitations in cognition, balance, or muscle tone may affect use and pose a safety risk. The product should only be used by people who can understand and follow the instructions.

To ensure continued safe use, the risk assessment should be repeated regularly over time.

Stable seated position when standing up

Ensure that the user is able to sit stably on the edge of the bed and have both feet firmly placed on the floor before standing up. This is necessary for the stand-up support to be used safely.

Bed compatibility

The product must be used on beds with a stable frame. The product must not be used on beds with skid bases, as these may slide or tip under load. In such cases, the skid base must be replaced with four stable bed legs prior to installation. When installing on a single bed (≤ 120 cm), the bed must always be positioned with one long side against a wall.

Beds where the mattress lacks sufficient edge stability and collapses under load near the edge of the bed are unsuitable for safe use, as the risk of entrapment may increase.


Use on adjustable beds

Let's Get Up must not be mounted on adjustable or variable beds, or used together with electric back rests. An exception applies only if the bed is locked in position and cannot be altered. In that case, check that the bed's settings cannot be changed accidentally by the user, as an entrapment risk may arise.

Technical data:


Tested according to ISO 60601-2-52:2009 for a maximum user weight of 150 kg.

Handle height	81 - 97 cm.
Frame bracket height	9 - 45 cm.
Floor plate length	59 cm.
Floor plate width	35 cm.
Weight	11.5 kg.

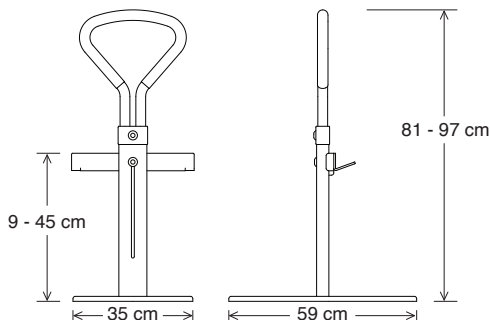MATERIAL

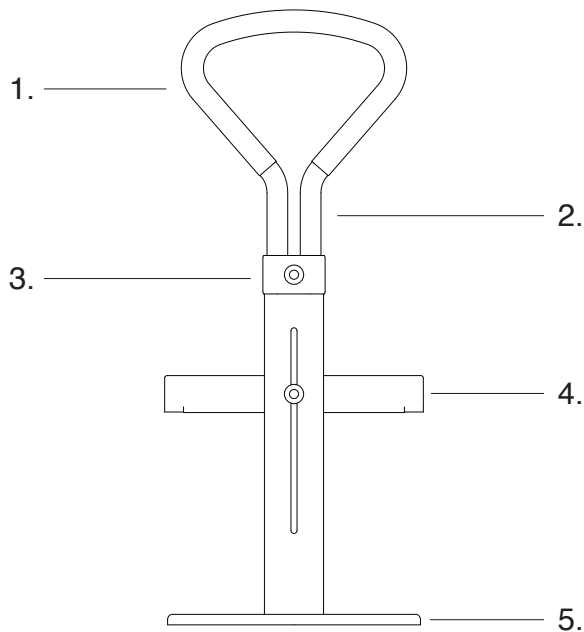1. Handtag - Skumgummi / Handle - Foam rubber
2. Handtagsrör - Aluminium / Handle tube - Aluminium
3. Höjjusteringskåpa - Plast / Height adjustment cover - Plastic
4. Sängfäste - Stål / Bed bracket - Steel
5. Golvplatta - Stål / Floor plate - Steel

Återvinning / Recycling

Denna produkt skall återvinnas i enlighet med nationella riktlinjer.
This product must be recycled in accordance with national guidelines.

www.trustcare.se

Trust Care AB
Skrittgatan 8 b, 213 77 Malmö
+46 40 15 61 20
info@trustcare.se
www.trustcare.se

June 2026 v6